

IF A VICTIM HAS COLLAPSED, OR HAS NO PULSE:



Cardiac Safe
COMMUNITY PROGRAM


FOLLOW THESE INSTRUCTIONS

- **OPEN RESCUE CASE DOOR**
- **AN ALARM WILL SOUND TO INDICATE AN EMERGENCY SITUATION EXISTS**
- **REMOVE THE DEFIBRILLATOR FROM THE CASE**

THIS WILL AUTOMATICALLY CALL **911 AND
EMERGENCY RESCUE PERSONNEL WILL BE
DISPATCHED**


- **AS A GOOD SAMARITAN YOU SHOULD:**

1 TURN
ON AED

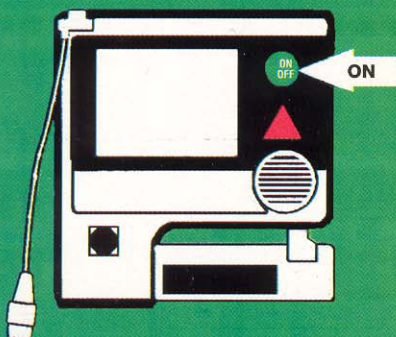

Press the Green "On/Off"
Button to turn system on.

2 FOLLOW
PROMPTS

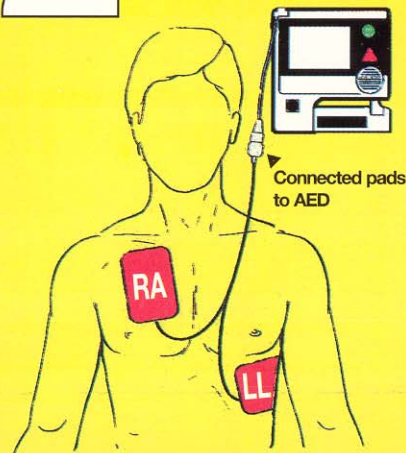

• Apply pads to patient's bare chest
• Connect Pads to AED

3 IF PROMPTED
PRESS SHOCK



**Do Not touch
the patient**

If prompted, press FLASHING
Red "SHOCK" Button